

Adult Advanced Cardiac Life Support


ARRHYTHMIA

ACLS PROTOCOL

ZOLL PRODUCT OPERATION

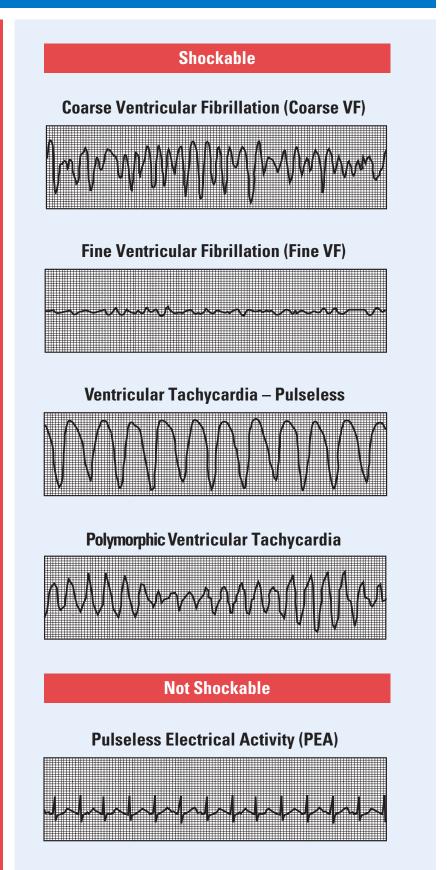

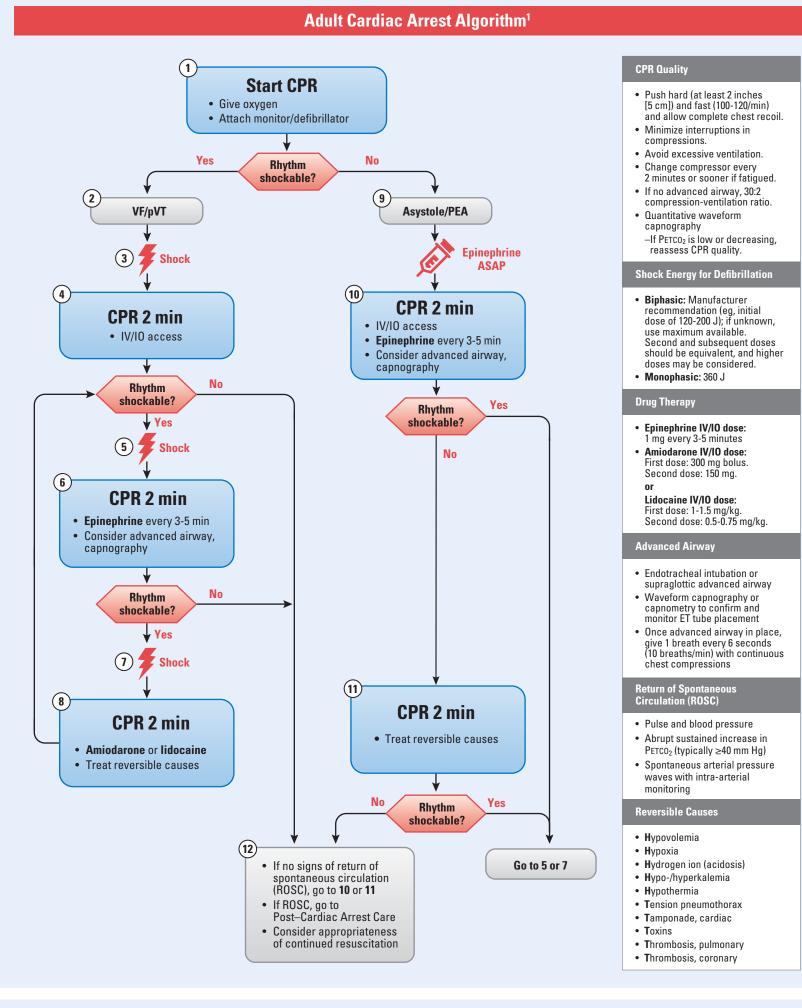


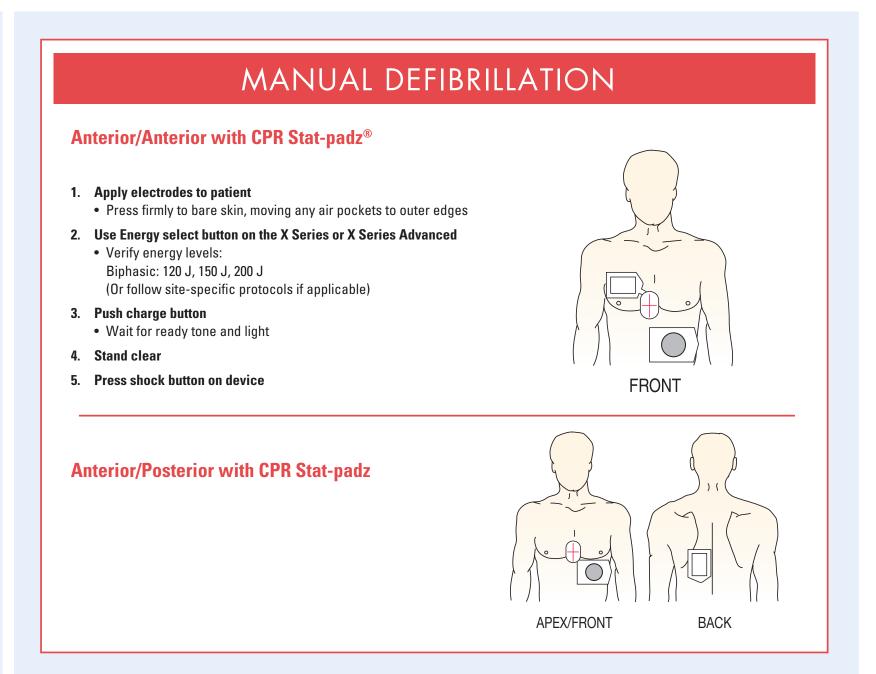


HIGH-QUALITY CPR IS VITAL


Components of high-quality CPR include:


- Ensuring chest compressions of adequate rate
- Ensuring chest compressions of adequate depth
- Allowing full chest recoil between compressions
- Minimizing interruptions in chest compressions
- Avoiding excessive ventilation

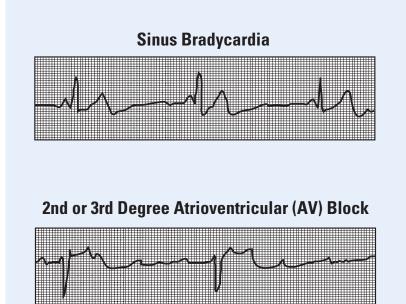


Asystole

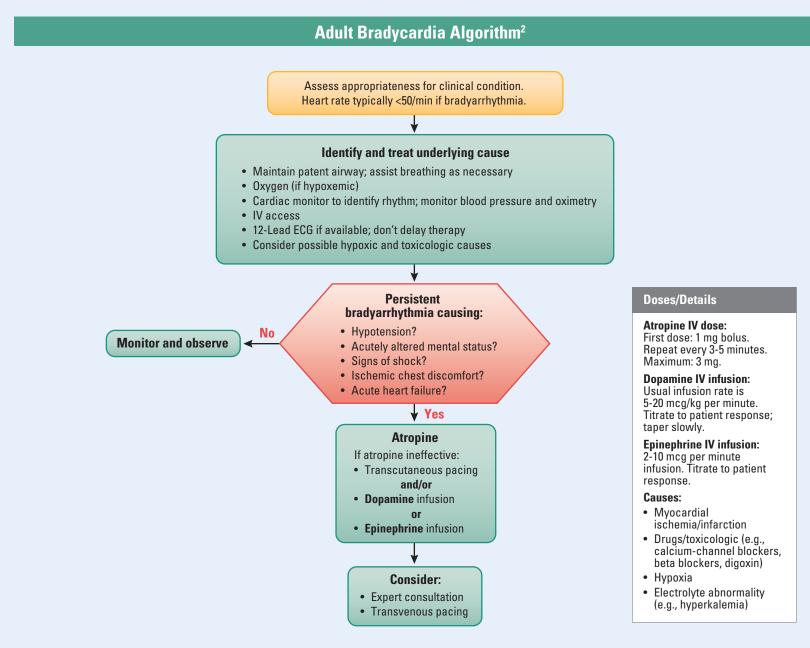


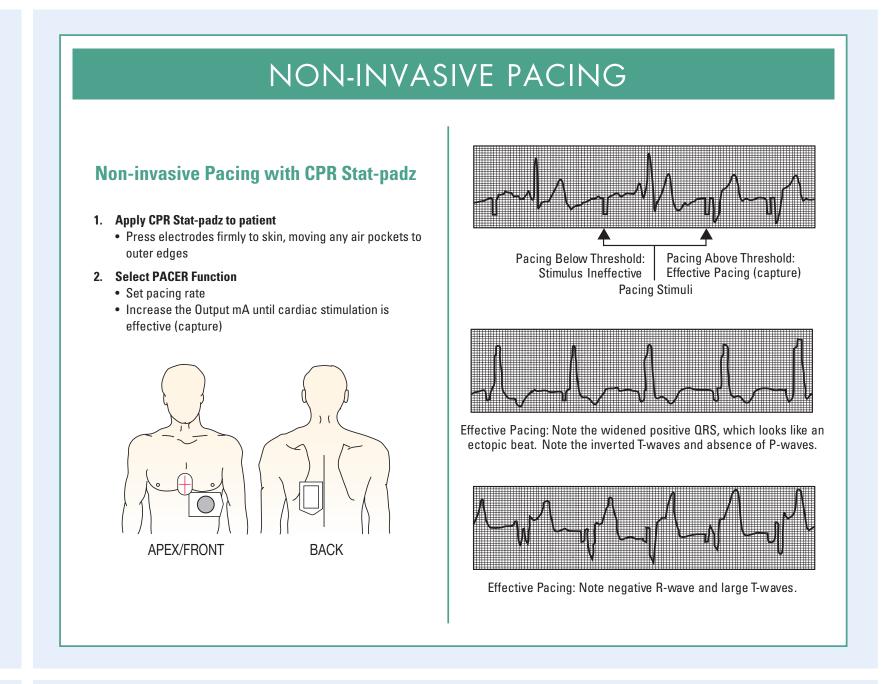

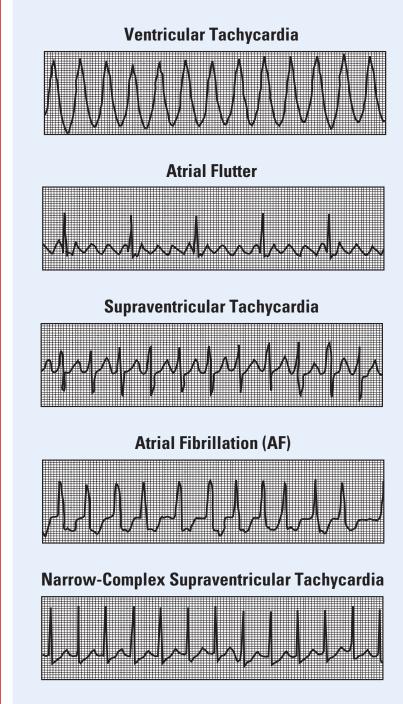

Stable Ventricular Tachycardia: Monomorphic

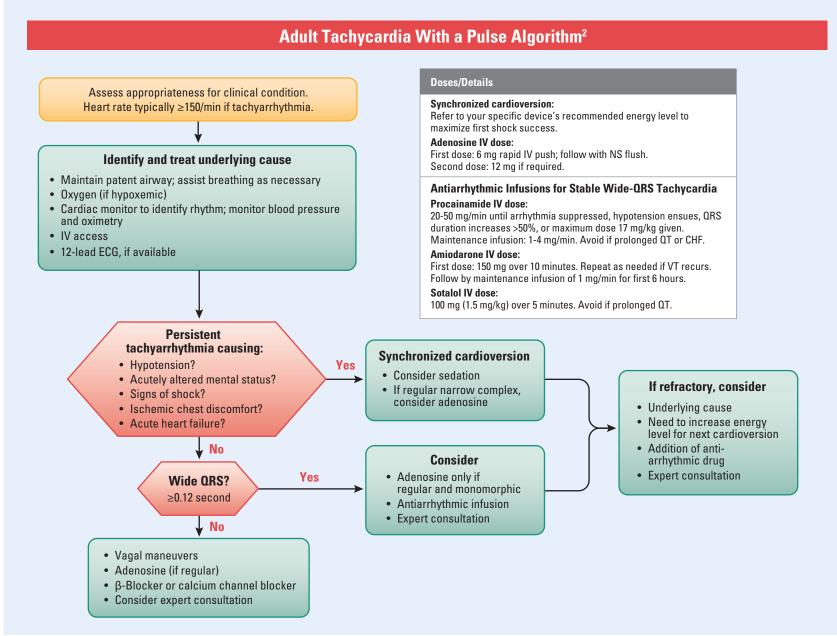

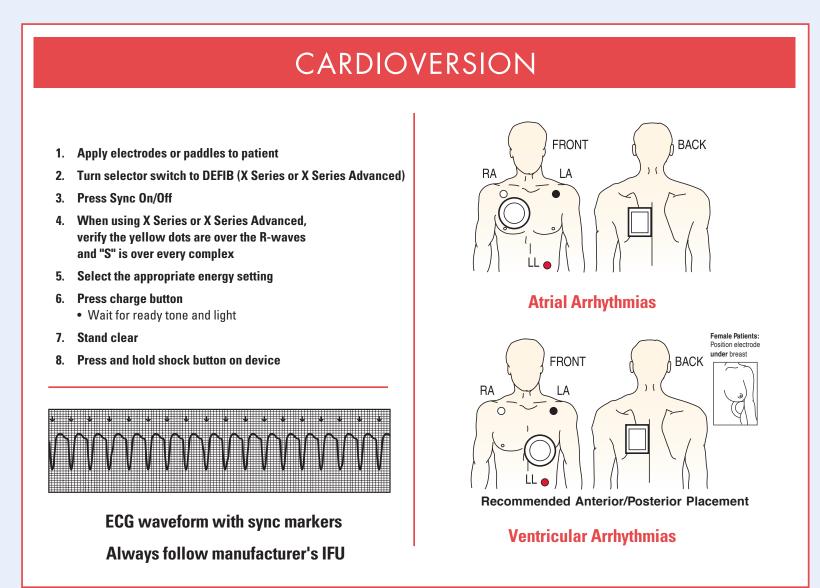

The information contained above summarizes the 2020 Adult Cardiac Arrest Protocol as published by the American Heart Association and is not meant as a substitute for comprehensive ACLS training. ZOLL Medical Corporation assumes no liability for changes or local interpretations of standard protocols. Some patients may require care not described herein. These guidelines should not limit treatment options.

©2021 ZOLL Medical Corporation. All rights reserved. X Series, X Serie Advanced, Stat-padz, and ZOLL are trademarks or registered trademarks of ZOLL Medical Corporation in the United States and/or other countries. All other trademarks are the property of their respective owners.

¹Reprinted with permission. Circulation.2020;142:S366-S468 ©2020 American Heart Association, Inc.

²Reprinted with permission. ACLS Provider Manual ©2020 American Heart Association, Inc.

ZOLL Medical Corporation 269 Mill Road Chelmsford, MA 01824-4105 978-421-9655 • 800-348-9011 www.zoll.com

