

ZOLL R Series ALS : Weekly Shock Test



*This document and other ZOLL related information can be found on the Health System INTRANET website under Resuscitation Committee website: "ZOLL Information"

Verify that pads are NOT connected to the patient

The ZOLL R Series ALS

In its ready state

Step #1

Verify pads are properly connected



Verify pads are not connected to patient

Step #2

Turn Mode Selector knob to DEFIB
(The Red Zone)



Step #3

Press ENERGY SELECT to set energy to 30 joules and then CHARGE



Step #4

Verify DEFIB 30J READY, but **do not** press press SHOCK



Step #5

Use ENERGY SELECT to set to 20 joules to disarm the previous charge



Step #6

Press ENERGY SELECT and go back to 30 joules and press charge



Step #7

When ready tone sounds and SHOCK button is illuminated, press SHOCK



Step #8

Look for the 30J TEST OK message (it displays for @ 2 seconds) and the print out indicating an OK test



Step #9

Look for the DEFIB PAD SHORT message: this is normal/OK and turn machine to OFF.

